

Contains Phosphoric Acid, and Glycolic Acid.

DIRECTIONS: Add 1 oz. (30 ml or 1 pump) to each quart (liter) of water. Aspirate solution through the high speed line, saliva ejector line and pour into the cuspidor. Aspirate 1 quart (1 liter) in each treatment room daily. First time users should add 2 oz (60 ml) to 1 quart (1 liter) of water for the first 5 days to remove accumulated debris.

FOR BEST RESULTS: Use every day. Lines and chairside trap (solids collector) will be free of all materials except amalgam. No manual cleaning will be necessary. To remove partial clogs, use PUREVAC double strength with hot water.

See recommended cleaning procedure at sultanhc.com

DANGER: May be corrosive to metals. Causes severe skin burns and eye damage. Do not breathe mists. Not for human consumption. Harmful if swallowed. Wash exposed skin thoroughly after handling. Wear protective gloves, protective clothing, eye protection, and face protection. Do not spill or leave PUREVAC on plastic laminate surfaces. When using on copper piping systems, flush with water following treatment. Store locked up. Dispose of contents and container in accordance with local and national regulations.

FIRST AID: IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water, or shower. Wash contaminated clothing before reuse. IF INHALED: Remove person to fresh air and keep comfortable for breathing. Immediately call a POISON CENTER, doctor, or physician. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. See SDS for complete safety information.

Manufactured for:



Sultan Healthcare
1301 Smile Way • York, PA 17404-0807 • USA
Toll Free: 800-637-8582 • Phone: 201-871-1232
Fax: 201-871-0321 • www.sultanhealthcare.com

 **EC REP** MDSS GmbH • Schiffgraben 41 • 30175 Hannover, Germany



Caution: consult SDS
at sultanhc.com

Made in U.S.A.