

DIRECTIONS FOR USE

Manual Cleaning: Mix 1oz. (30mL) of Zymex Enzymatic Cleaner Concentrate to 1 gallon (3.8L) of water. Soak instruments as needed. We recommend a 5 minute presoak for instruments contaminated with blood. Immerse instruments completely and aspirate solution into any channels and lumens. After soak period, visually inspect for cleanliness and rinse thoroughly with clean water.

Reprocessors: Mix ½ oz. (15mL) of concentrate per gallon (3.8L) to the reprocessor reservoir or attach container to the reprocessor's automated dispensing unit.

Ultrasonic: Add 1oz. (30mL) of concentrate per gallon (3.8L) of water. Soak for 2-3 minutes at 90-110°F (32-43°C). Process instruments according to manufacturer's directions.

Washer Sterilizers/Decontaminators: Attach concentrate container to unit's automatic dispensing unit.

Diluted product must be discarded daily or sooner if contaminated.

Sultan Healthcare recommends that after using Zymex Enzymatic Cleaner Concentrate a thorough rinse and rough dry should be completed, especially if instruments are to be chemically disinfected. Always consult manufacturer's recommendations.

Danger!

Precautions: Users must always wear protective eye wear, gloves and clothing. If accidental eye contact with Zymex Enzymatic Cleaner Concentrate occurs, flush eyes for 10 minutes and seek immediate medical attention. IF INHALED: remove person to fresh air and keep comfortable for breathing. If experiencing respiratory symptoms: Call a POISON CENTER. See SDS for complete safety information. Contains Subtilisin, may cause allergy or asthma symptoms or breathing difficulties if inhaled.

Storage and Disposal: Store at room temperature. To dispose of enzymatic solution, flush thoroughly with large quantities of water into sewage disposal system in accordance with federal, state and local regulations. Rinse container thoroughly before discarding. Do not reuse container.

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a healthcare professional.



Consult SDS at
Sultanhc.com



+D6870021390FG1-



Made in U.S.A.

0021390L, R2-031414

LOT/EXP - Zebra printed