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Versa-Temp[®] Temporary Crown and Bridge Resin

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EC REP MDSS GmbH • Schiffaraben 41 • 30175 Hannover. Germanv

Versa-Temp® Temporary Crown and Bridge Resin

Directions for Use

Indications For Use

Versa-Temp® Temporary Crown and Bridge Resin is a 2-part chemically-cured composite resin material. It is indicated for the fabrication of temporary dental crowns and bridges.

Contraindications

This product contains methacrylates, amines, terpenes, benzoyl peroxide and BHT (butylated hydroxytoluene). Do not use with patients who have known allergies or sensitivities to these materials.

Warnings

Keep this material out of reach of children. For professional dental use only. Observe the contraindications indicated in the previous section.

Precautions

Avoid contact with eyes and skin. Wear eye protection and gloves while using. Store unused material away from direct light and between 39°F – 73°F (4°C – 23°C). If restoring a tooth with a composite core, lubricate the core with glycerine or any other suitable water-based lubricant.

Adverse Reactions

Uncured product may cause mild eye and skin irritation. Prolonged or repeated contact with uncured product may cause skin sensitization.

Directions for use

- Fabricate a matrix using addition-cured silicone putty, silicone impression material (Genie®VPS Impression material is recommended), thermoplastic resin or alginate, that reflects the contours of the planned provisional restoration. This restoration can be made directly from the patient’s pre-operative oral tissues, or it can be made from a model of the proposed restorations.
- Remove the cap from the **Versa-Temp**® Temporary Crown and Bridge Resin cartridge and discard. Place the cartridge into a dispensing gun and bleed a small amount of material from the cartridge orifice ensuring product is flowing equally from both ports. Affix a light blue mixer tip to the cartridge and lock it by turning the hub 90° clockwise until a “click” is felt. The cartridge is now ready for use.
- Complete the clinical preparation and establish tissue retraction and hemostasis. If a composite core is present, lubricate the core with glycerine or a suitable water-based lubricant to prevent adhesion to the composite material (the use of petroleum-based lubricants may interfere with proper setting and final strength of the material).
- Dispense a very small amount of material from the mixing tip and discard. Fill the matrix in the areas to be restored with **Versa-Temp**® Temporary Crown and Bridge Resin.

Insert the resin-filled matrix into the patient’s mouth within 30 seconds of mixing (or onto a lubricated cast of the prepared teeth if using an indirect fabrication method). Make sure the matrix is correctly positioned and firmly hold the matrix in place. Leave the mixing tip affixed on the cartridge until the next use.

- Versa-Temp**® Temporary Crown and Bridge Resin undergoes a “rubber” phase approximately 60 seconds from initial mixing. It is important to remove the matrix during this phase. Carefully remove the matrix (the provisional should remain in the matrix) from the mouth and allow it to fully cure in the matrix. If the provisional restoration remains on the tooth, gently remove it using an instrument and allow it to cure on the bench. The tips of cotton pliers on either side of the restoration will allow it to be gently lifted occlusally/incisally. **Do not** allow the provisional restoration to fully cure on the tooth.
- Versa-Temp**® Temporary Crown and Bridge Resin should fully cure in 4 minutes from the start of mixing. After 4 minutes, the provisional restoration can be trimmed and the margins and contours shaped using carbide burs, diamond disks or diamond rotary instruments. If desired, the operator can remove the air-inhibited surface layer with ethyl alcohol and a cotton pad prior to trimming.
- Small voids can be repaired either with flowable composite or with additional **Versa-Temp**® Temporary Crown and Bridge Resin. Note that if the air-inhibited layer has been removed by solvent or trimming, apply a small amount of **GlossCote**™ to the surface (do not light cure) prior to applying the repair material.
- Adjust the proximal and occlusal contacts of the provisional restoration.

- Polish the provisional restoration prior to cementation. Suitable polishing methods are abrasive-impregnated rubber points or flour of pumice on a rag wheel followed by acrylic polish. The surface of the provisional restoration should appear smooth with a matte finish. If a high polish is desired, apply **GlossCote**™ to the entire surface of the restoration and light cure for 30 seconds (take care not to apply **GlossCote**™ on the margins).
- Cement the restoration with a non-eugenol cement such as **Sensi-Temp**™ Resin.
- Re-check the occlusion and interproximal cleansability prior to dismissing the patient.

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- Stellen Sie aus Silikonkitt, Silikon-Abdruckmasse (wir empfehlen Genie© VPS), einem Thermoplast oder Alginat eine die Konturen des geplanten Provisoriums wiedergebende Matrix her. Diese Abformung kann direkt vom präoperativen Mundgewebe des Patienten oder von einem Modell der geplanten Restaurationen genommen werden.