

Xylol (Xylenes)

DIRECTIONS FOR USE

REF – 21401 – 1oz/30mL

For dental use only.

Xylol is supplied as a general solvent that can be used for cleaning up certain hardened cements and materials from tools and mixing equipment. This product is for professional use only. **IT IS NOT INTENDED FOR HUMAN USE OR CONTACT.** Observe all warnings and precautions on this sheet, the product label and the MSDS sheet.

CAUTION

Do not take internally. May be fatal if swallowed. Vapor is harmful. Avoid breathing vapor. Avoid skin contact.

FLAMMABLE

Keep away from open flame or heat sources. The flash point of xylol is 24°C (75°F).

WARNING

This product is not intended for human use or contact.



Sultan Healthcare
1301 Smile Way • York, PA 17404-0807 • USA
Toll Free: 800-637-8582 • Phone: 201-871-1232
Fax: 201-871-0321 • www.sultanhealthcare.com

Made in the USA
0021401DF, R1-092414